



Promoting Health by  
Preventing Error.



# Medication Crushers

**Powdercrush**  
**CalmPress**  
**Silent Knight**  
**Crush Pouches and Cups**

## Powdercrush®

**ITEM NUMBER: 39-011PE** - Available individually



The Powdercrush® automatic medication crusher. No more painful grinding and pounding. Simply push a button. The battery-powered Powdercrush® does the job for you – quickly, cleanly and painlessly.

**Width: 8.25 cm (3.25 in)**

**Height: 9.50 cm (3.74 in)**

**Length: 27.90 cm (10.98 in)**

**Weight: 2.70 kg (5.95 lbs.)**



1. Place pills to be crushed in a Manrex supplied disposable medication pouch. Fold over the top of the pouch to keep the powder inside the pouch.



2. With one hand, insert the disposable medication pouch filled with pills in the slot on top of the machine. Press down on the start button to crush meds.



3. Typically, 4-5 seconds will produce the desired results. For a finer powder operate for a longer period.

## CalmPress®

**ITEM NUMBER: MBM395BL** - Available individually



CalmPress® is a silent, effective manual crushing device, through the use of medication pouches.

**Width: 10.16 cm (4.00 in)**

**Height: 10.16 cm (4.00 in)**

**Length: 25.40 cm (10.00 in)**

**Weight: 2.20 kg (4.85 lbs.)**



1. Place your meds in the Manrex disposable medication pouch, lift the handle and insert the pouch into CalmPress crushing slot.



2. Press down on handle to crush medication starting at the bottom of the pouch. Gradually lower the pouch into the crusher as you work the handle up and down.



3. Remove pouch or repeat if a finer crush is required.



### Silent Knight™

The Silent Knight™ is a silent yet powerful medication crusher. Tablets are crushed to a fine powder that will dissolve quickly in feeding tubes. This Silent Knight™ medication crusher requires minimum force resulting in fewer wrist and arm injuries.

**ITEM NUMBER: SK0500** - Available individually



### PharmaPouch

The new, economical PharmaPouch eliminates cross contamination. During crushing, the tablets are held in this strong durable plastic pouch that doesn't tear. No clean up required after use.

**ITEM NUMBER: 39-1000** - Box of 1000

**ITEM NUMBER 39-8000** - Box of 8000



### Pill Crusher Pouch

The Pill Crusher Pouch eliminates cross contamination. During crushing, the tablets are held in this strong, durable plastic pouch that doesn't tear. No clean up required after use.

**ITEM NUMBER: QCP1000** - Box of 1000

**ITEM NUMBER: QCP8000** - Box of 8000



### Crush Cups

The Crush Cups are 1/2 oz. plastic medication cups that are typically used with the Dual Cup dispensers that fit the UltraGlide cart.

**ITEM NUMBER: MBM7365** - Case of 5040 1/2 oz. Cups



### Graduated Crush Cups

The 1 oz. Graduated Cups are plastic medication cups that are typically used with the 1 oz. and Dual Cup dispensers.

**ITEM NUMBER: 7371** - Bag of 100 1 oz. Cups