



Promoting Health by
Preventing Error.



Medication Crushers

Powdercrush
CalmPress
Silent Knight
Crush Pouches and Cups

Powdercrush®

ITEM NUMBER: 39-011PE - Available individually



The Powdercrush® automatic medication crusher. No more painful grinding and pounding. Simply push a button. The battery-powered Powdercrush® does the job for you – quickly, cleanly and painlessly.

Width: 8.25 cm (3.25 in)

Height: 9.50 cm (3.74 in)

Length: 27.90 cm (10.98 in)

Weight: 2.70 kg (5.95 lbs.)



1. Place pills to be crushed in a Manrex supplied disposable medication pouch. Fold over the top of the pouch to keep the powder inside the pouch.



2. With one hand, insert the disposable medication pouch filled with pills in the slot on top of the machine. Press down on the start button to crush meds.



3. Typically, 4-5 seconds will produce the desired results. For a finer powder operate for a longer period.

CalmPress®

ITEM NUMBER: MBM395BL - Available individually



CalmPress® is a silent, effective manual crushing device, through the use of medication pouches.

Width: 10.16 cm (4.00 in)

Height: 10.16 cm (4.00 in)

Length: 25.40 cm (10.00 in)

Weight: 2.20 kg (4.85 lbs.)



1. Place your meds in the Manrex disposable medication pouch, lift the handle and insert the pouch into CalmPress crushing slot.



2. Press down on handle to crush medication starting at the bottom of the pouch. Gradually lower the pouch into the crusher as you work the handle up and down.



3. Remove pouch or repeat if a finer crush is required.



Silent Knight™

The Silent Knight™ is a silent yet powerful medication crusher. Tablets are crushed to a fine powder that will dissolve quickly in feeding tubes. This Silent Knight™ medication crusher requires minimum force resulting in fewer wrist and arm injuries.

ITEM NUMBER: SK0500 - Available individually



PharmaPouch

The new, economical PharmaPouch eliminates cross contamination. During crushing, the tablets are held in this strong durable plastic pouch that doesn't tear. No clean up required after use.

ITEM NUMBER: 39-1000 - Box of 1000

ITEM NUMBER 39-8000 - Box of 8000



Pill Crusher Pouch

The Pill Crusher Pouch eliminates cross contamination. During crushing, the tablets are held in this strong, durable plastic pouch that doesn't tear. No clean up required after use.

ITEM NUMBER: QCP1000 - Box of 1000

ITEM NUMBER: QCP8000 - Box of 8000



Crush Cups

The Crush Cups are 1/2 oz. plastic medication cups that are typically used with the Dual Cup dispensers that fit the UltraGlide cart.

ITEM NUMBER: MBM7365 - Case of 5040 1/2 oz. Cups



Graduated Crush Cups

The 1 oz. Graduated Cups are plastic medication cups that are typically used with the 1 oz. and Dual Cup dispensers.

ITEM NUMBER: 7371 - Bag of 100 1 oz. Cups



Crushield

With its smooth, stainless-steel roller running over rounded ridges, Crushield quickly and effortlessly crushes pills into dust with just a few strokes of its ergonomic handle! This smooth roller is gentle on the crushing pouch and combined with the Crushield's unique multi-layer pouch construction, the risk of punctures is greatly reduced. The Crushield's springloaded handle provides a simple push and release dual action... crushing pills on the down stroke, and crushing pills as the super strong spring returns the handle to the ready position at the end of each gliding stroke. Ensures reduced risk of sore wrists or shoulders common with other manual crushers. The Crushield's open design ensures cleaning, when necessary, is effective and hygienic as all parts can be accessed and wiped down without removing covers or shields. The Crushield is made of stainless steel, a preferred material in health care facilities and anodized aluminum for durability that will last for years.

ITEM NUMBER: MAX-001 - Available individually



Liquid Crushing Pouch

Crushield offers a major improvement in convenience when loading tablets into the Liquid EnFit compatible pouch! Simply unscrew the top, drop the pills in, attach an EnFit syringe filled with liquid to the syringe receptacle on top and inject the liquid once the medications are crushed. Shake vigorously to blend and then withdraw the contents back into the syringe. Attach the security seal cap to the pouch once empty, reducing the risk of liquid spills or contamination.

ITEM NUMBER: MAX-002 - Box of 600



Zipper Top Security Seal Pouch

Reduce the risk of exposure to powdery medication pill dust by crushing in Crushield's security sealed zip-top pouch. To use, simply open the zip-top pouch, place the pills inside, gently expel the air in the pouch and then re-seal the zip-top. The zip-top seal is designed for maximum sealing effectiveness. After crushing the pills, gently remove the perforated top section to retrieve the newly crushed medications without having to reopen the zip-top, reducing the risk of pill dust exposure.

ITEM NUMBER: MAX-003 - Box of 2000



Tear Away Top Pouch

Suitable for use in routine crushing situations, the Crushield easy fold pouch makes crushing simple, clean and safe. Its size allows it to work with a wide range of crushers on the market, but its perforated fold over top ensures pill dust is not expelled while crushing and its tear away top makes removing the pulverized pills easy and efficient.

ITEM NUMBER: MAX-004 - Box of 4000